## **BEST PRODUCT**

# Extract with confidence & luxate atraumatically

Familiar LM™ ergonomics and tactile sensitivity also in extracting instruments

- Ergonomic design
- comfortable to hold and easy to rotate
- Secure LM-ErgoTouch<sup>™</sup> surface
- light, non-slip grip
- Well-balanced
- controlled grip
- Lightweight
- less strain on the hands
- Clear color-coding
- easy identification
- LM-DuraGradeMAX® supersteel
- strong and sharp blades
- Tight seam at blade juncture
- hygienic



Best Product 2014, Dental Product Shopper


### LM-LIFTOUT™





Luxating instruments for atraumatic extraction.

LM-LiftOut™ instruments allow tooth extraction to be done as atraumatically as possible. This is important in order to enable rapid healing and future implant placement. The thin tip of the instrument is introduced into the periodontal space and slowly advanced toward the apex of the root while twisting the instrument gently back and forth. LM-LiftOut™ is not suitable for use as an elevator.

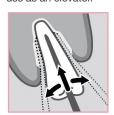







## LM-SLIMLIFT<sup>TM</sup>





Luxating instruments designed for extremely narrow places.

LM-SlimLift™ instruments are especially designed to reach places you cannot go with traditional luxating instruments. It allows tooth extraction to be done as atraumatically as possible. This is important in order to enable rapid healing. The extremely fine tip of the instrument is introduced into the very narrow periodontal space. The instrument is slowly advanced towards the apex of the root while turning the instrument gently back and forth. The LM-SlimLift™ is not suitable for use as an elevator.







LM 812630

LM 812630T

LM 812830T








LM 812640 LM 812640T LM-SlimLift™ S5 LM 812650 LM 812650T

LM 812650T

The extremely slim design of blade allows for the most atraumatic extractions.

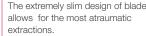

#### **MAINTENANCE**



## Cassette LM 810001





LM™ extraction instruments are supplied in a convenient cassette that protects both the instrument and the handler during the maintenance cycle. The cassette keeps the instrument from puncturing the sterilization pouch, so sterility is more likely to be assured.

- The cassette can be color-coded
- The cassette is compatible with LM-ServoMax™ tray

## Sharpening stone LM 818002

Sharpen the extractors occasionally using the round sharpening stone as a simple and quick way to keep them in top condition and always ready to use.

- Suitable for LM-SlimLift™, LM-LiftOut™, LM-TwistOut™ and LM-RootOut™ instruments



LM 812640T

## LM-TWISTOUT™





Elevators for forceful extraction.

LM-TwistOut™ instruments are suitable for tooth extraction in situations where strong force or torque must be applied.









